



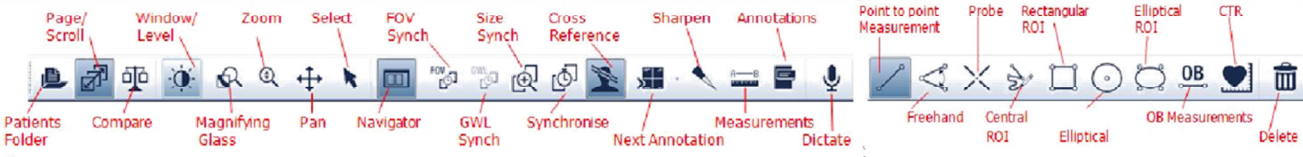





# Quick Guide to *Centricity PACS-UV*

Getting Started	<p>UV will open up and display the worklist for the first tab configured on the system, usually set up for <b>TODAY</b>. Any of the Title blocks e.g. Patient Name can be clicked on to sort the worklist order.</p> <p> A green note pad and tick indicates that there is a radiology report for the patient.</p>
Search	<p>Clicking on the  button will bring up the search page.</p> <p>Selecting <b>Last Search</b> — bottom left hand side — will load the last search criteria entered.</p>
Opening the Study	<p>A single left click on the study will open all the images.</p>
Viewing the Report	<p>Click on the Dictate / Microphone  button in the toolbar (top left hand side) to open the report.</p>
Compare Studies	<p>Select the cases you wish to compare from the worklist by clicking in the square on the far left of the patient's name. A tick will appear in the boxes.</p> <p>Click on the compare button . The older case will have a dotted line around it when viewed. Use the Navigator to load images next to each other if necessary.</p>
Mouse Controls	<p>The right mouse button defaults to <u>windowing</u> image centre and width (changing contrast &amp; brightness). Roll the mouse wheel to scroll through images. When in <u>zoom</u> mode, clicking and holding the mouse wheel activates the <u>pan</u> function.</p>
Toolbars	<p>Annotation and measurement toolbars are expanded when clicking on the appropriate icon.</p>  <p> Select next hanging protocol, clicking on the drop down arrow will show a list of all applicable hanging protocols.</p> <p> The chronological timeline displays, representing a tile for each study grouped by dates associated with the patient selected.</p>
Closing a Study	<p> Clicking this icon closes the study. This will close the study and not change the status or save any changes to the images.</p>
Shortcut Keys	<p>0 = Rotate Image          6 = Reset Zoom          R = Reset Window Level (brightness)          V = Invert Image (black on white to see ET tubes, CVP lines, etc.)          Spacebar = Next Preset Window Level (i.e. bone, lung, soft tissue)          Home = First Image          End = Last Image          Page Up / Down = Fast Forward / Backwards through Images</p>
	<p>CR = General X-rays e.g. Chest; Chest mobile/portable          CT = CT scans          MR = MRI scans          MG = Mammography          RF = Screening and Angiography          US = Ultrasound</p>